

## Use book

- REF** -34257 Oxygen Reservoir 1000 ml with valve - single use  
-34258 Oxygen Reservoir 2500 ml with valve - single use  
-34275 Oxygen Reservoir bag - 2500ml

### Accessories

- 34279 Reservoir Valve

### Product Description:

The device designed for use in a breathing circuit as a reservoir for breathing gas and to assist, control, or monitor a patient's ventilation. This product is intended for use by qualified medical personnel.

### Usage instruction:

1. Eye check for clear airflow passage prior to use.
2. Connect the breathing bag to breathing circuit.
3. Make sure the connection is compatible and fit firmly.






### Warning:

1. The product should be placed in a dry and clean place and sealed conditions.
2. The operator should be trained and know the skill in CPR resuscitation.

### Precautions:

- Disposable product-single use only
- The patient should be closely monitored by qualified medical personnel when using this device.
- This product is hygienically packed.
- Read this instruction manual carefully before use.
- Store in clean dry conditions. Away from heat and light.

## Simboli - Symbols

	Prodotto conforme alla Direttiva Europea Product complies with European Directive
	Leggere attentamente le istruzioni per l'uso Please read instructions carefully
	Leggere e seguire attentamente le istruzioni per l'uso Read instructions carefully
	Conservare al riparo dalla luce solare Keep away from sunlight
	Data di fabbricazione Date of manufacture
	Data di scadenza (vedi scatola / bustina) Expiration date (see box / package)
	Codice prodotto Product code
	Fabbricante Manufacturer
	Conservare in luogo fresco ed asciutto Keep in a cool, dry place
	Numero di lotto (vedi scatola / bustina) Lot number (see box / package)
	Dispositivo monouso, non riutilizzare Disposable device, do not re-use

